

TECHNICAL DATA SHEET



AMPri Handelsges. mbH • Benzstraße 16 • 21423 Winsen/Luhe, Germany • Tel. 04171 84 80-0 • www.ampri.de • info@ampri.de

Sharp disposables container 1,5 l"Series PBS Article No.: L-09301



General Data:	Disposable container for sharp and cutting objects disposal. Item designed for low volumes of waste (care tray, trolley, domestic care, doctor´s office)
Material:	Plastic material 100 % virgin Polypropylene
Colour:	Yellow body and red lid in order to signal the dangerous content. Materials coloured with masterbatch without heavy metals and cadmium. Materials compliant to REACH.
Thickness:	Bottom and sides: 1,9 mm – Lid and closure: 1,9 mm
Shape:	Truncated cone
Weight:	155 g
Max Load:	850 g
Nominal capacity:	1,500 Lt.
External dimensions:	Ø 140 x 165h mm
Base dimensions:	Ø 102
Certifications:	Compliant to AFNOR NF X-30-511 Compliant to NF Mark Compliant to ADR – UN Approved Compliant to ISO 23907 – Kite Mark Compliant to FDA K071517 Compliant to TRBA 250
Manufacturer:	AP medical

TECHNICAL DATA SHEET



AMPri Handelsges. mbH • Benzstraße 16 • 21423 Winsen/Luhe, Germany • Tel. 04171 84 80-0 • www.ampri.de • info@ampri.de

INSTRUCTIONS

► Lid assembly:

Firmly press on all the perimeter, until the lid clicks on the body



► Temporary closure

The container could be temporarily closed using the cap upside down.

At this stage, closure is temporary



IMPORTANT!

Before use, press on the anti-reflux fins in order to create a funnel-shaped opening. This device eases the insertion of the waste and creates a barrier to prevent from leakage



► Permanent closure

The container could be closed permanently applying pressure on the cap until it clicks on the lid.

At this stage, closure is permanent



USE:

► During use, close the cap in temporary closure position



► Close the container permanently once full



- Large opening to ease the disposal of scalpels, test tubes, etc.
- Autoclave Suitable to autoclave at 134 ° for 18 minutes
- Once filled, the containers have to be disposed through incineration, releasing no toxic emissions

TECHNICAL DATA SHEET



AMPri Handelsges. mbH • Benzstraße 16 • 21423 Winsen/Luhe, Germany • Tel. 04171 84 80-0 • www.ampri.de • info@ampri.de

► Needles removal:

Use the “V” removal device to separate needle and syringe

► Insulin pens removal:

Use the “marguerite” device to unscrew the insulin pen



AVAILABLE HOLDERS:

► Trolley/wall bracket

► Special plastic tray

► Desk suction cups holder



Advantages:

- Lid provided with safe closure device, permanent closure, prevents from waste reflux
- Ease to use and to handle
- Sharp and cutting objects resistant
- Handles available
- Lid provided with needles removal devices
- Suitable to autoclave (compliant to NFX 30-500 even after sterilization)
- Maximum filling level indicated
- Several holders available
- Irreversible permanent closure
- To be disposed through incineration, releasing no toxic emissions

Safety information:

- Do not use the product before fitting the lid
- Do not attempt to rescue objects already in the container
- Respect the filling level indicated: 80 % of real capacity
- Use the container only with appropriate lid
- Do not try to open the container once permanently and irreversibly sealed
- follow the instructions of use to ensure the safety of the container
- Container to dispose according to the existing law
- Use handles to move the container